

DiroCHEK[®]

The Gold Standard



Synbiotics raises
The Gold Standard
to a higher level.

DiroCHEK is recognized as *The Gold Standard* for accuracy in leading labs, universities and clinics worldwide.

As the longest and largest selling heartworm antigen test kit, DiroCHEK has earned the confidence of veterinarians worldwide.

Canine & Feline Heartworm Antigen Detection

- **Even better!** New conjugate - Enhanced stability - Optimized specificity.
- **Flexible:** Test one sample at a time or batches
- **Economical:** Batch testing saves time and money
- **Easy:** Fewer steps than other heartworm microwell tests

- **Accurate:**

JW McCall, et.al. Study to determine the performance of commercially available canine heartworm antigen test kits. *Proceedings of the AAVP 45th Annual Meeting*, July 2000.

	Negative	1 female	2 females	≥ 3 females
Test	n = 100	n = 21	n = 20	n = 57
DiroCHEK [®]	100%	85.7%	95%	100%
PetChek [®] HTWM PF	100%	76.2%	85%	100%
Solo Step [™] CH Batch	100%	71.4%	95%	96.5%

- **Available:** Phone your favorite Synbiotics Distributor

Go for the Gold Standard: DiroCHEK

PetChek[®] is a registered trademark of IDEXX Laboratories, Inc.
Solo Step[™] is a trademark of Heska Corp.

©2000 Synbiotics Corporation, #4000, Rev. 6/03 Printed in USA



11011 Via Frontera, San Diego, CA 92127
800-228-4305

www.synbiotics.com

Synbiotics Heartworm Screening Guide

History:

Pet is under 6 months of age: Start on monthly preventive

Pet's history and heartworm status is unknown

Pet is over 6 months of age and about to begin heartworm preventive

Pet is resuming preventive for the coming heartworm season

Pet is currently on a preventive

Tests:

DOGS & CATS
Heartworm Antigen Test

Results:




Synbiotics
Technical Services
800-228-4305

Confirm Results:
Is result consistent with history & clinical evidence? Repeat test, perform microfilaria test, radiographs, and/or ultrasound.

Results confirmed?

No

Yes

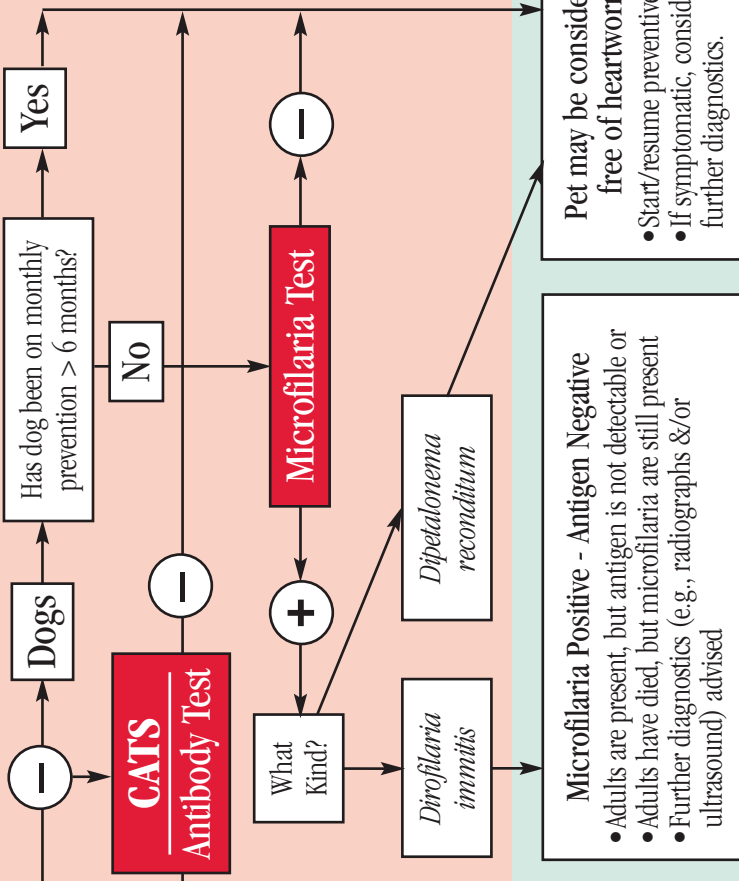

Conclusions:

Retest in 3-6 mos.
• Contact Synbiotics Technical Services for consultation

Pet has heartworm infection
• Determine appropriate therapy

Microfilaria Positive - Antigen Negative
• Adults are present, but antigen is not detectable or
• Adults have died, but microfilaria are still present
• Further diagnostics (e.g., radiographs &/or ultrasound) advised

Pet may be considered free of heartworm
• Start/resume preventive
• If symptomatic, consider further diagnostics.



DiroCHEK® The Gold Standard

- Heartworm antigen test kit
- Microwell format