
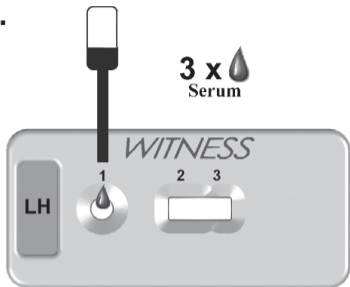
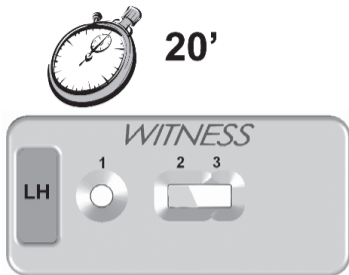
1.

2.

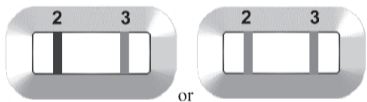
3.

$LH \geq 1\text{ng/ml}$

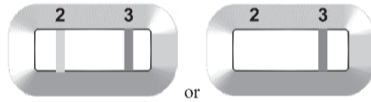3.

$LH < 1\text{ng/ml}$

# ***WITNESS LH***

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## **GENERAL INFORMATION**

Identification of the luteinizing hormone (LH) surge provides the most accurate means of canine ovulation timing. It may be used for routine breeding situations and is especially recommended in those instances where there are factors present that could adversely affect conception rates. These include: chilled/extended semen breedings, frozen semen breedings, bitches with a history of infertility, breedings with stud dogs with low semen quality, limited access to the stud dog.

Once an increase of LH is identified, SYNBIOTICS recommends that daily serum samples should be tested for progesterone levels. If the detected increase in LH was indeed the actual pre-ovulatory surge, progesterone levels should rise, usually reaching a value above 2ng/mL within 3 days, and then stay elevated.

Identification of bitches and queens that have had their ovaries removed is possible through the detection of LH. Serum concentrations of LH increase after gonadectomy in dogs and cats. Because LH concentrations in serum are <1ng/mL in sexually intact bitches and queens, except during LH peak at estrus, it may be assumed that high values in bitches and queens are indicative that their ovaries have been removed.

## **TEST INDICATION**

- **OVULATION TIMING (Bitches)**

WITNESS LH, when used in conjunction with in-hospital progesterone testing, identifies the pre-ovulatory LH surge, and thus the time of ovulation. Ovulation occurs 2 days after the LH surge : the oocytes then require an additional 2- days to mature, and will live for about 48-72 hours.

- **DISTINGUISH BETWEEN OVARIECTOMIZED AND SEXULALLY INTACT (Bitches or Queens)**

WITNESS LH may also be used to distinguish between ovariectomized and sexually intact bitches or queens.

## **TEST PRINCIPLE**

WITNESS LH test provides an accurate, semi-quantitative measurement of canine and feline luteinizing hormone. The test is simple to perform and rapidly provides information for the veterinary clinician. The test is an immunochromatographic assay that uses gold-conjugated antibodies to give a visual line in the presence of luteinizing hormone.

WITNESS LH allows you to identify the occurrence of the LH surge. Since the LH surge may occur within a 24-hour period, it is crucial that daily serum samples are tested. Blood should be drawn every day at approximately the same time beginning the 4th or 5th day of proestrus, or when vaginal cytology approaches 50% cornification (cornification is defined as cells with a roughened, angulated border with or without a nucleus). A progesterone test should also be performed on the first day of testing to confirm a baseline progesterone level. If testing is started after the onset of estrus, it is likely that the LH surge will have already occurred and cannot be identified in retrospect.

A positive result occurs when a line appears in the test area which is of similar or greater intensity than the control line. When this occurs, the LH level in the serum sample is greater than 1 nanogram per milliliter. Control bands located at the end of the reading windows (3) ensure that the test was performed correctly.

## **SPECIMEN INFORMATION**

- The test can **ONLY** be performed on **SERA**.
- The samples should not be hemolyzed or lipemic.

### **Storage**

Serum samples should preferably be tested immediately after collection.

If testing is further delayed, sera should be kept refrigerated (+2°C - 8°C) for up to 24 hours.

For prolonged storage, sera should be kept frozen (-20°C). Do not thaw and refreeze.

## **KIT CONTENTS**

- A. 6 pouches, each containing 1 test device and desiccant.
- C. Instructions for use.
- D. 6 pipettes

## **GENERAL PRECAUTIONS**

1. Do not use components after the expiration date.
2. Store the test kit at 2°C – 25°C. Do not freeze.
3. Use the test immediately after opening the sealed pouch (within 10 minutes)
4. Avoid touching or damaging membrane windows (1), (2), (3).
5. The WITNESS device should be placed on a flat, horizontal surface while performing the test.
6. Use a separate pipette for each sample.
7. Hold pipette vertically when dispensing sample.
8. For veterinary use only.

## **TEST PROCEDURE AND RESULTS**

### **1. Sample application**

- Tear open a pouch provided and place the test device on a flat horizontal surface.
- Holding the provided pipette vertically, transfer three drops of sample to sample well (1).
- Leave the test device flat during migration of sample/reagent complex through the reading windows.

### **2. Reading test**

After 20 minutes, observe the presence or absence of pink bands in reading windows (2) and (3).

### **3. RESULTS**

#### **1. Validation**

The test is validated if a pink band is present in reading window (3).

#### **2. Interpretation**

- No band in reading window (2) or a band which is less intense than the control line (3), means that the LH value is less than 1ng/mL
- One band in reading window (2) which is of similar or greater intensity than the control line (3) means that the LH value is greater than 1ng/mL.

#### **Note :**

- A positive result in a bitch or queen that is not in estrus indicates that the ovaries have been removed. Repeat the test in 2 hours with a fresh sample to confirm the results. A negative result provides excellent evidence that the bitch or queen is intact.
- During proestrus, small, pulsatile fluctuations in LH may occur, while progesterone remains at low, baseline levels. Progesterone rises, however, after the pre-ovulatory surge of LH. By the third day post LH peak, the majority of bitches will evidence a rise in progesterone levels above 2ng/mL level. Daily progesterone testing after a positive WITNESS LH test result allows simple differentiation between small fluctuating increases in LH during proestrus and the true, pre-ovulatory LH surge. If there is no confirmation of progesterone rise within 3 days of a positive WITNESS LH test, it may be assumed that the positive test result was due to a baseline LH fluctuation.

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