

WITNESS[®] Relaxin

Canine/ Feline Relaxin Hormone Diagnostic Test

Early Detection of Pregnancy

Reliable Assessment of Pregnancy Status

Results in 10 Minutes

Five Tests per Kit



12200 N. Ambassador Drive, Suite 101 • Kansas City MO 64163
800-228-4305 • 816-464-3500 • Fax: 816-464-3521 • www.synbiotics.com

2, rue Alexander Fleming • 69367 LYON Cedex 07 • France
+33 4 72 76 11 11 • Fax: +33 4 72 76 11 10



WITNESS[®] Relaxin

Canine/Feline Relaxin Hormone Diagnostic Test





Witness[®] Relaxin detects the hormone relaxin produced by the developing placenta(s) as early as day 20 after the luteinizing hormone (LH) surge. Factors such as breed, size of the bitch/queen and litter size may influence the level of serum relaxin. Although approximately 80% of pregnant bitches/queens are detected between 20 and 28 days post-LH surge, some may not be detected as positive until day 31. Negative results need to be re-tested at least seven days later.

Facts:

- Relaxin is secreted primarily by the placenta and is the only hormone specifically associated with canine and feline pregnancy.
- Levels of relaxin remain elevated throughout gestation and decline quickly following whelping/queening or abortion.

Witness[®] Relaxin offers an early, inexpensive and reliable means for determining pregnancy status.

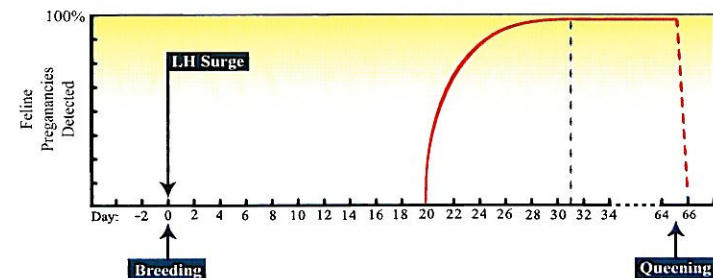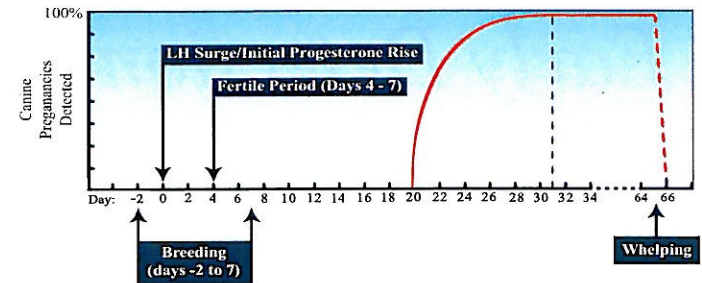

When It's Time to Test *(Serum or Plasma Sample Required)*

- 1** Apply 2 drops of sample using the pipette
- 2** Add 2 Drops of Buffer
- 3** Wait 10 Minutes
- 4** Interpret Results